

MANUFACTURER'S 60-DAY LIMITED WARRANTY

The manufacturer warrants that all parts and components are free of defects in materials and workmanship for 60 days from the date the product is received. This warranty is valid only in accordance with the conditions set forth below:

1. Normal wear and tear are not covered by this warranty. This warranty applies to the consumer use only, and is void when the product issued in a commercial or institutional setting.
2. The warranty extends only to the original consumer purchaser and is not transferable. In addition, proof of purchase must be demonstrated.
3. This limited warranty is the only written or express warranty given by the manufacturer. Any implied warranty of merchantability or fitness for a particular purpose on this product is limited in duration to the duration of this warranty. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.
4. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

**Perfect
Smile™
veneers**

We are very proud of the design and quality of our Perfect Smile. This product has been manufactured to the highest standards. Should you have any problem, our friendly customer service staff is here to help you.

973.287.5128

PerfectSmileVeneers.com



Distributed By:
Tristar Products, Inc.
Fairfield, NJ 07004
© 2017 Tristar Products, Inc.
Made in China

PSV_WARRANTY-INSERT_TP_ENG_V4_170601

**Perfect
Smile™
veneers**

Thank you!

**Congratulations on Your Purchase of
Perfect Smile Veneers!**

**Custom Fit Perfect Smile Veneers
in 3 Easy Steps:**

One Size Fits All



Warranty Information Inside.

Custom Fit Perfect Smile Veneers in 3 Easy Steps:

Step 1: Place Perfect Smile in boiling water for 2 minutes.

- When impression material on the back of Perfect Smile turns clear and soft, remove with a metal spoon or fork. If impression material is not soft, re-heat water.

Caution: Perfect Smile will be hot. Wait 10-15 seconds before handling.

Impression material will turn clear when it is ready.

Step 2: Press firmly against your teeth for the perfect, custom fit.

- Press Perfect Smile firmly against your teeth. Gently push the impression material against the back of your top row of teeth. Also try to push down the material from the roof of your mouth.
- PLEASE NOTE:** Do not keep your Perfect Smile in your mouth for more than 30 seconds when molding it to your teeth for the first time. You do not want the impression material to harden while on your teeth.
- After removing Perfect Smile from your mouth, insert it in a glass of cold water to cool before using. The impression material will turn white when it is properly hardened.

Step 3: Insert Perfect Smile over your teeth & show off your million dollar smile!

- Try Perfect Smile on. Feel free to re-mold as many times as you want until you get the perfect, custom fit.
- Congratulations! You're ready to begin wearing Perfect Smile.

Watch our step-by-step **how-to video** on how to get a perfect, custom fit at perfectsmileveneers.com/customfit

This product is **ONLY** intended to enhance your appearance, and is not a DENTURE or a DENTAL DEVICE.



WARNING:
CHOKING HAZARD - SMALL PARTS

IMPORTANT SAFEGUARDS

- Do not use if you have gum disease, loose or broken teeth, severe tooth decay or extensive dental work. Remove Perfect Smile within 30 seconds of first application before it hardens into its permanent shape. Failure to follow these instructions can result in damage to your teeth or mouth.
- Do not use tobacco products while wearing Perfect Smile.
- Avoid hot beverages while wearing Perfect Smile.
- Do not eat or chew gum while wearing Perfect Smile.
- Do not use Perfect Smile over braces or retainers.
- Remove Perfect Smile while sleeping.
- Do not wear Perfect Smile if it is damaged in any way.
- Remove Perfect Smile if you feel any discomfort.
- Do not wear Perfect Smile if you have any cuts or other types of irritation on the inside of your mouth.
- Perfect Smile is not a toy and should not be used by children.
- Do not substitute Perfect Smile for a mouth guard.
- Do not wear Perfect Smile while playing sports or during strenuous activities.
- Do not wear Perfect Smile for longer than eight hours at a time.

CARING FOR YOUR PERFECT SMILE

All you need to do to care for your Perfect Smile is soak it in mouthwash for about an hour, once a day. We recommend soaking overnight while you sleep. You may also use a toothbrush to clean Perfect Smile every time you're finished wearing it.

In order for this product to stay firm in the mouth, a person needs to have some existing tooth structure.

For millions of people who simply can't afford expensive dental work, Perfect Smile is an affordable way to look better and restore your confidence. The thin construction is designed for comfort and a

natural appearance. Perfect Smile is intended only for temporary use when you want to enhance your appearance, but is comfortable enough to be worn all day long.

This product is not a "denture" or "dental device." Perfect Smile is intended only to enhance your appearance. **It is not designed to wear while eating or to be used as a denture. Do not sleep or eat while wearing it.**

For more information, go to PerfectSmileVeneers.com.

FREQUENTLY ASKED QUESTIONS

- How long should it take to make the mold to my teeth?**
Do not keep your Perfect Smile in your mouth for more than 30 seconds when molding it to your teeth for the first time. You do not want the impression material to harden while on your teeth.
- Can I eat or drink while wearing Perfect Smile?**
Do not wear Perfect Smile when eating. Although you may drink, avoid hot beverages.
- Can I sleep while wearing Perfect Smile?**
Do not wear Perfect Smile when sleeping.
- Can I wear Perfect Smile more than once?**
Perfect Smile is designed to be worn frequently.
- Does Perfect Smile come in different sizes?**
Perfect Smile is One-Size-Fits-All. It will custom-fit around anyone's teeth.
- What if Perfect Smile doesn't fit after molding?**
See fitting instructions.
- What is Perfect Smile made of?**
Perfect Smile is made of custom-moldable state-of-the-art polymer.
- Can I wear Perfect Smile over braces?**
No, Perfect Smile is not made to fit over braces.
- Is it difficult to speak while wearing Perfect Smile?**
No. Although it may take a little time to get used to speaking while wearing Perfect Smile, you'll be ready to go after a few minutes.